

## Kreatech™ FISH probes


### Product Information Sheet

17Q002B550  
ERBB2 (17q12)  
Red  
50 µl

**DANGER**



**FORMAMIDE**

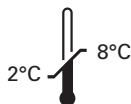


**Kreatech Biotechnology B.V.**  
Vlierweg 20  
1032 LG Amsterdam  
The Netherlands  
[www.LeicaBiosystems.com](http://www.LeicaBiosystems.com)

#### Analyte Specific Reagent

Analytical and performance characteristics are not established

Published November 2016

PI-17Q002B550\_D1.0

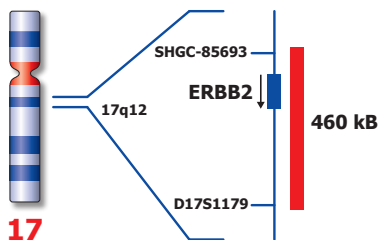

## Kreatech™ ERBB2 (17q12) FISH probe

- Introduction:** The **ERBB2 (17q12)** specific FISH probe is optimized to detect the ERBB2 (Her2/neu) gene region at region 17q12.
- Critical region (red):** The **ERBB2 (17q12)** specific FISH probe is direct-labeled with Platinum*Bright*™550.
- Reagent:** The ERBB2 (17q12) FISH probe is a direct-labeled DNA probe provided at two times the concentration than the amount of DNA probe used in the QC test procedure.
- Kreatech FISH probes are REPEAT-FREE™ and therefore do not contain Cot-1 DNA. Hybridization efficiency is increased and background, due to unspecific binding, is highly reduced.**
- Patterns:** Two red (2R) signals will identify the non-aberrant ERBB2 loci.

	Signal Pattern
Expected Signals	2R

**Warning and precautions:** In case of emergencies check SDS sheets for medical advice. SDS sheets may be obtained by either contacting Leica Technical Support or visiting [www.LeicaBiosystems.com](http://www.LeicaBiosystems.com). DNA probes contain formamide which is a teratogen; do not inhale or allow skin contact. Wear gloves and a lab coat when handling DNA probes. All materials should be disposed of according to your institution's guidelines for hospital waste disposal.

### Reagent Storage and

### Handling:

Store at 2-8 °C. Reagents should not be used after the expiration date on the vial label.

### TECHNICAL SUPPORT

Technical support is available at [www.LeicaBiosystems.com/service-support/technical-support/](http://www.LeicaBiosystems.com/service-support/technical-support/) or toll free at 800-248-0123 or via e-mail: [kreatech-support@leicabiosystems.com](mailto:kreatech-support@leicabiosystems.com).

### CUSTOMER SERVICE

Kreatech probes may be ordered through Leica Customer Service toll free at 800-248-0123 or order via e-mail: [purchase.orders@leica-microsystems.com](mailto:purchase.orders@leica-microsystems.com).