

Novocastra™ Lyophilized Mouse Monoclonal Antibody CD20

Product Code: NCL-CD20-7D1

Intended Use	For In Vitro Diagnostic Use: This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.
Specificity	Human CD20 antigen.
Clone	7D1
Ig Class	IgG1
Antigen Used for Immunizations	Prokaryotic recombinant fusion protein corresponding to the C-terminal intracellular domain of the transmembrane CD20 molecule.
Hybridoma Partner	Mouse myeloma (p3-NS1-Ag4-1).
Preparation	Lyophilized tissue culture supernatant containing 15 mM sodium azide. Reconstitute with the volume of sterile distilled water indicated on the vial label.
Effective on Frozen Tissue	Yes
Effective on Paraffin Wax Embedded Tissue	Yes (using the high temperature antigen unmasking technique: see overleaf).
Recommendations on Use	Immunohistochemistry: Typical working dilution 1:100–1:200. 60 minutes primary antibody incubation at 25 °C. High temperature antigen unmasking technique. Standard ABC technique. Western Blotting: Typical working dilution 1:50–1:100. Effective in indirect flow cytometry.
Positive Controls	Immunohistochemistry: Tonsil. Western Blotting: Tonsil.
Staining Pattern	B lymphocyte membrane-associated staining in germinal centres and mantle zone areas of tonsil.
Storage and Stability	Store unopened lyophilized antibody at 4 °C. Under these conditions, there is no significant loss in product performance up to the expiry date indicated on the vial label. The reconstituted antibody is stable for at least two months when stored at 4 °C. For long term storage, it is recommended that aliquots of the antibody are frozen at -20 °C (frost-free freezers are not recommended). Repeated freezing and thawing must be avoided. Prepare working dilutions on the day of use.
General Overview	The CD20 antigen is a non-glycosylated phosphoprotein of approximately 35 kD. The CD20 antigen is expressed on normal human B cells and is thought to act as a receptor during B cell activation and differentiation. CD20 antigen has been reported to be detected on normal B cells from peripheral blood, lymph node, spleen, tonsil and bone marrow.
General References	Coles F B, Cartun R W and Pastuszak W T. <i>Modern Pathology</i> . 1: 274–278 (1988). Linder J, Yüling Y, Armitage J O, et al.. <i>Modern Pathology</i> . 1: 29–34 (1988). Davey F R, Gatter K C, Ralfkiaer E, et al.. <i>American Journal of Pathology</i> . 129: 54–63 (1987). Ishii Y, Takami T, Yuasa H, et al.. <i>Clinical and Experimental Immunology</i> . 58: 183–192 (1984).



Instructions for Use

High Temperature Antigen Unmasking Technique for Immunohistochemical Demonstration on Paraffin Sections

1. Cut and mount sections on slides coated with a suitable tissue adhesive.
2. Deparaffinize sections and rehydrate to distilled water.
3. Place sections in 0.5% hydrogen peroxide/methanol for 10 minutes (or use other appropriate endogenous peroxidase blocking procedure). Wash sections in tap water.
4. Heat 1500 mL of the recommended unmasking solution (0.01 M citrate buffer, pH 6.0 (or Epitope Retrieval Solution, RE7113) unless otherwise indicated overleaf) until boiling in a stainless steel pressure cooker. Cover but do not lock lid.
5. Position slides into metal staining racks (do not place slides close together as uneven staining may occur) and lower into pressure cooker ensuring slides are completely immersed in unmasking solution. Lock lid.
6. When the pressure cooker reaches operating temperature and pressure (after about 5 minutes) start a timer for 1 minute (unless otherwise indicated on the data sheet).
7. When the timer rings, remove pressure cooker from heat source and run under cold water with lid on. DO NOT OPEN LID UNTIL THE INDICATORS SHOW THAT PRESSURE HAS BEEN RELEASED. Open lid, remove slides and place immediately into a bath of tap water.
8. Wash sections in TBS* buffer (pH 7.6) for 1 x 5 minutes.
9. Place sections in diluted normal serum (or RTU Normal Horse Serum) for 10 minutes.
10. Incubate sections with primary antibody. Use Antibody Diluent RE7133 (where available).
11. Wash in TBS buffer for 2 x 5 minutes.
12. Incubate sections in an appropriate biotinylated secondary antibody.
13. Wash in TBS buffer for 2 x 5 minutes.
14. Incubate slides in ABC reagent (or RTU streptavidin/peroxidase complex).
15. Wash in TBS buffer for 2 x 5 minutes.
16. Incubate slides in DAB or other suitable peroxidase substrate.
17. Wash thoroughly in running tap water.
18. Counterstain with hematoxylin (if required), dehydrate and mount.

Solutions

0.01 M CITRATE BUFFER (pH 6.0) or RE7113 (where available).

Add 3.84 g of citric acid (anhydrous) to 1.8 L of distilled water. Adjust to pH 6.0 using concentrated NaOH. Make up to 2 L with distilled water.

1 mM EDTA (pH 8.0) or RE7116 (where available).

Add 0.37 g of EDTA (SIGMA product code E-5134) to 1 litre of distilled water. Adjust pH to 8.0 using 1.0 M NaOH.

20 mM TRIS/ 0.65 mM EDTA/ 0.005% TWEEN (pH 9.0) or RE7119 (where available).

Dissolve 14.4 g Tris (BDH product code 271197K) and 1.44 g EDTA (SIGMA product code E-5134) to 0.55 L of distilled water. Adjust pH to 9.0 with 1 M HCl and add 0.3 mL Tween 20 (SIGMA product code P-1379). Make up to 0.6 L with distilled water. This is a 10x concentrate which should be diluted with distilled water as required (eg 150 mL diluted with 1350 mL of distilled water).

* In most applications, 10 mM phosphate, 0.15 M NaCl, pH 7.6 (PBS) can be used instead of 50 mM Tris, 0.15 M NaCl, pH 7.6 (TBS).

Safety Note

To ensure the correct and safe use of your pressure cooker, PLEASE READ MANUFACTURER'S INSTRUCTIONS.