

# Novocastra™ Lyophilized Mouse Monoclonal Antibody Estrogen Receptor (beta)

**Product Code: NCL-ER-beta**

## Analyte Specific Reagent

<b>Clone</b>	EMR02
<b>Ig Class/Isotype</b>	IgG1
<b>Ig Concentration</b>	See vial label.
<b>Presentation</b>	Dialyzed and lyophilized tissue culture supernatant containing 15 mM sodium azide. Reconstitute with the volume indicated on the vial label.
<b>Specificity</b>	Human estrogen receptor (beta). Epitope mapped to a 17 amino acid region specific to the wild type form of the C-terminus. This epitope is not present in ERβcx.

## Precautions and Warnings

Analyte Specific Reagent. Analytical and performance characteristics are not established. This reagent has been prepared from the supernatant of cell culture. As it is a biological product, reasonable care should be taken when handling it.

The molarity of sodium azide in this reagent is 15 mM. Sodium azide (NaN<sub>3</sub>) is a highly toxic chemical in pure form. Although at 15 mM it is not classified as hazardous, a build-up of NaN<sub>3</sub> may react with lead and copper plumbing to form highly explosive metal azides. To dispose of this reagent, flush with large volumes of water to prevent azide building up in the plumbing.

## Statement of Quality

Each lot of reagent has been quality controlled by immunohistochemistry.

## Storage and Stability

Store unopened lyophilized antibody at 2–8 °C. Under these conditions, there is no significant loss in product performance up to the expiry date indicated on the vial label. The reconstituted antibody is stable for at least two months when stored at 2–8 °C. For long term storage, it is recommended that aliquots of the antibody are frozen at -20 °C (frost-free freezers are not recommended). Repeated freezing and thawing must be avoided. Prepare working dilutions on the day of use. If reagents are stored under any conditions other than those specified, the conditions must be verified by the user.

## General References

Department of Health, Education and Welfare, National Institute for Occupational Safety and Health, Rockville, MD.

"Procedures for the decontamination of plumbing systems containing copper and or lead azides." 1976.

Clinical Laboratory Improvement Amendments of 1988: Final Rule 57CFR7163. February 28, 1992.

Rees M L, Marshall I, McIntosh G G, et al.. *Hybridoma and Hybridomics*. 23 (1): 11–18 (2004).

Omoto Y, Kobayashi S, Inoue S, et al.. *European Journal of Cancer*. 38: 380–386 (2002).

Scobie G A, Macpherson S, Millar M R, et al.. *Steroids*. 67: 985–992 (2002).

Skirris G P, Parkes A T, Limer J L, et al.. *Journal of Pathology*. 197: 155–162 (2002).

Pavao M and Traish A M. *Steroids*. 66: 1–16 (2001).

Chu S, Marners P, Burger H G, et al.. *The Journal of Clinical Endocrinology & Metabolism*. 85 (3): 1200–1205 (2000).

Fuqua S A W, Schiff R, Parra I, et al.. *Cancer Research*. 59: 5425–5428 (1999).

