

Novocastra™ Lyophilized Mouse Monoclonal Antibody Osteopontin

Product Code: NCL-O-PONTIN

Intended Use	For In Vitro Diagnostic Use: This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.
Specificity	Human osteopontin.
Clone	OP3N
Ig Class	IgG1
Antigen Used for Immunizations	Prokaryotic recombinant protein corresponding to a portion of the C-terminus of the osteopontin molecule.
Hybridoma Partner	Mouse myeloma (p3-NS1-Ag4-1).
Preparation	Lyophilized tissue culture supernatant containing 15 mM sodium azide. Reconstitute with the volume of sterile distilled water indicated on the vial label.
Effective on Frozen Tissue	Not fully evaluated.
Effective on Paraffin Wax Embedded Tissue	Yes (using the high temperature antigen unmasking technique: see overleaf).
Recommendations on Use	Immunohistochemistry: Typical working dilution 1:50–1:100. High temperature antigen unmasking technique. 60 minutes primary antibody incubation at 25 °C. Standard ABC technique. Western Blotting: Not recommended.
Positive Controls	Immunohistochemistry: Gall bladder.
Staining Pattern	Cytoplasmic.
Storage and Stability	Store unopened lyophilized antibody at 4 °C. Under these conditions, there is no significant loss in product performance up to the expiry date indicated on the vial label. The reconstituted antibody is stable for at least two months when stored at 4 °C. For long term storage, it is recommended that aliquots of the antibody are frozen at -20 °C (frost-free freezers are not recommended). Repeated freezing and thawing must be avoided. Prepare working dilutions on the day of use.
General Overview	Osteopontin is a 34 kD extracellular matrix protein with a cell binding domain. Other molecules which share this domain include fibronectin, vitronectin and a variety of other extracellular proteins that bind members of the integrin family of cell surface receptors. Osteopontin was originally identified as a major component of the non-collagenous organic bone matrix, however, it has subsequently been demonstrated in a wide range of normal adult tissues and body fluids. It is a multifunctional protein involved in bone mineralization, cell adhesion and cell migration. Other studies using in situ hybridization and immunohistochemistry have shown staining in the epithelium of gastrointestinal tract, gall bladder, pancreas, urinary and reproductive tracts, lung, salivary and sweat glands. Ganglion cells in the bowel also express osteopontin as do macrophages, T cells and NK cells upon activation.

General References

Stawowy P, Blaschke F, Pfautsch P, et al.. The European Journal of Heart Failure. 4: 139–146 (2002).
Zhang J, Takahashi K, Takahashi F, et al.. Cancer Letters. 171: 215–222 (2001).
Nau G J, Chupp G L, Emile J-F, et al.. American Journal of Pathology. 157 (1): 37–42 (2000).
Crawford H C, Matrisian L M and Liaw L. Cancer Research. 58: 5206–5215 (1998).
Denhardt D T and Guo X. The FASEB Journal. 7: 1475–1482 (1993).
Brown L F, Berse B and Van De Water L. Molecular Biology of the Cell. 3: 1169–1180 (1992).



Instructions for Use

High Temperature Antigen Unmasking Technique for Immunohistochemical Demonstration on Paraffin Sections

1. Cut and mount sections on slides coated with a suitable tissue adhesive.
2. Deparaffinize sections and rehydrate to distilled water.
3. Place sections in 0.5% hydrogen peroxide/methanol for 10 minutes (or use other appropriate endogenous peroxidase blocking procedure). Wash sections in tap water.
4. Heat 1500 mL of the recommended unmasking solution (0.01 M citrate buffer, pH 6.0 (or Epitope Retrieval Solution, RE7113) unless otherwise indicated overleaf) until boiling in a stainless steel pressure cooker. Cover but do not lock lid.
5. Position slides into metal staining racks (do not place slides close together as uneven staining may occur) and lower into pressure cooker ensuring slides are completely immersed in unmasking solution. Lock lid.
6. When the pressure cooker reaches operating temperature and pressure (after about 5 minutes) start a timer for 1 minute (unless otherwise indicated on the data sheet).
7. When the timer rings, remove pressure cooker from heat source and run under cold water with lid on. DO NOT OPEN LID UNTIL THE INDICATORS SHOW THAT PRESSURE HAS BEEN RELEASED. Open lid, remove slides and place immediately into a bath of tap water.
8. Wash sections in TBS* buffer (pH 7.6) for 1 x 5 minutes.
9. Place sections in diluted normal serum (or RTU Normal Horse Serum) for 10 minutes.
10. Incubate sections with primary antibody. Use Antibody Diluent RE7133 (where available).
11. Wash in TBS buffer for 2 x 5 minutes.
12. Incubate sections in an appropriate biotinylated secondary antibody.
13. Wash in TBS buffer for 2 x 5 minutes.
14. Incubate slides in ABC reagent (or RTU streptavidin/peroxidase complex).
15. Wash in TBS buffer for 2 x 5 minutes.
16. Incubate slides in DAB or other suitable peroxidase substrate.
17. Wash thoroughly in running tap water.
18. Counterstain with hematoxylin (if required), dehydrate and mount.

Solutions

0.01 M CITRATE BUFFER (pH 6.0) or RE7113 (where available).

Add 3.84 g of citric acid (anhydrous) to 1.8 L of distilled water. Adjust to pH 6.0 using concentrated NaOH. Make up to 2 L with distilled water.

1 mM EDTA (pH 8.0) or RE7116 (where available).

Add 0.37 g of EDTA (SIGMA product code E-5134) to 1 litre of distilled water. Adjust pH to 8.0 using 1.0 M NaOH.

20 mM TRIS/ 0.65 mM EDTA/ 0.005% TWEEN (pH 9.0) or RE7119 (where available).

Dissolve 14.4 g Tris (BDH product code 271197K) and 1.44 g EDTA (SIGMA product code E-5134) to 0.55 L of distilled water. Adjust pH to 9.0 with 1 M HCl and add 0.3 mL Tween 20 (SIGMA product code P-1379). Make up to 0.6 L with distilled water. This is a 10x concentrate which should be diluted with distilled water as required (eg 150 mL diluted with 1350 mL of distilled water).

* In most applications, 10 mM phosphate, 0.15 M NaCl, pH 7.6 (PBS) can be used instead of 50 mM Tris, 0.15 M NaCl, pH 7.6 (TBS).

Safety Note

To ensure the correct and safe use of your pressure cooker, PLEASE READ MANUFACTURER'S INSTRUCTIONS.