# Bond™ Ready-To-Use Primary Antibody Mast Cell Tryptase (10D11)

**Catalog No: PA0019**

Leica Biosystems Newcastle Ltd  
Balliol Business Park West  
Benton Lane  
Newcastle Upon Tyne NE12 8EW  
United Kingdom  
☎ +44 191 215 4242

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## Instructions for Use

Please read before using this product.

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## Check the integrity of the packaging before use.

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Kontroller, at pakken er ubeskadiget før brug.

# Bond™ Ready-To-Use Primary Antibody Mast Cell Tryptase (10D11)

## Catalog No: PA0019

### Intended Use

*This reagent is for in vitro diagnostic use.*

Mast Cell Tryptase (10D11) monoclonal antibody is intended to be used for the qualitative identification by light microscopy of human mast cell tryptase in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using an automated Bond™ system.

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

### Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using Bond Reagents" in your Bond user documentation). Mast Cell Tryptase (10D11) primary antibody is a ready to use product that has been specifically optimized for use with Bond Polymer Refine Detection. The demonstration of human mast cell tryptase is achieved by first, allowing the binding of Mast Cell Tryptase (10D11) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with an automated Bond system, reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

### Reagents Provided

Mast Cell Tryptase (10D11) is a mouse anti-human monoclonal antibody produced as a tissue culture supernatant, and supplied in Tris buffered saline with carrier protein, containing 0.35% ProClin™ 950 as a preservative.

Total volume = 7 mL.

### Clone

10D11.

### Immunogen

Prokaryotic recombinant protein corresponding to the unbound region of the human mast cell tryptase molecule.

### Specificity

Human mast cell tryptase.

### Subclass

IgG2b.

### Total Protein Concentration

Approx 10 mg/mL.

### Antibody Concentration

Greater than or equal to 0.03 mg/L as determined by ELISA.

### Dilution and Mixing

Mast Cell Tryptase (10D11) primary antibody is optimally diluted for use on a Bond system. Reconstitution, mixing, dilution or titration of this reagent is not required.

### Materials Required But Not Provided

Refer to "Using Bond Reagents" in your Bond user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using a Bond system.

### Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of Mast Cell Tryptase (10D11) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user<sup>1</sup>.

### Precautions

- This product is intended for in vitro diagnostic use.
- The concentration of ProClin™ 950 is 0.35%. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' Web site, [www.LeicaBiosystems.com](http://www.LeicaBiosystems.com).
- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions<sup>2</sup>. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.

- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

## Instructions for Use

Mast Cell Tryptase (10D11) primary antibody was developed for use on an automated Bond system in combination with Bond Polymer Refine Detection. The recommended staining protocol for Mast Cell Tryptase (10D11) primary antibody is IHC Protocol F. No pretreatment is recommended.

## Results Expected

### Normal Tissues

Clone 10D11 detected mast cells in a variety of tissues including liver, pancreas, lung, heart, spleen, skin, testis, cervix, uterus, ovary, breast, prostate, thyroid, thymus, tonsil, submandibular gland, peripheral nerve, bone marrow, pituitary, mesothelium and all areas of the GI tract. (Total number of cases stained = 54).

### Tumor Tissues

Clone 10D11 gave specific staining in a case of systemic mastocytosis and detected mast cells in a variety of other cases. No tumor specific staining was seen in the cases evaluated, including squamous cell carcinomas, adenocarcinomas, sarcomas, melanomas and lymphomas. (Total number of cases stained = 45).

Mast Cell Tryptase (10D11) is recommended for use as part of a panel of antibodies for the detection of mast cells in normal and neoplastic tissues, and may be useful for the identification of systemic mastocytosis.

## Product Specific Limitations

Mast Cell Tryptase (10D11) has been optimized at Leica Biosystems for use with Bond Polymer Refine Detection and Bond ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

## Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

## Further Information

Further information on immunostaining with Bond reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using Bond Reagents" in your Bond user documentation.

## Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Rygol B, Kyrz-Krzemien S, Pajak J et al. Tryptase- and chymase-positive mast cells as a possible prognostic factor in patients with Hodgkin's lymphoma. POLSKIE ARCHIWUM MEDYCYN WĘWNETRZNEJ (Polish Archive of Domestic Medicine) 2007; 117:1–2.
5. Anaf V, Chapron C, El Nakadi I et al. Pain, mast cells, and nerves in peritoneal, ovarian, and deep infiltrating endometriosis. Fertility and Sterility. 2006; 86(5):1336–1343.

ProClin™ 950 is a trademark of Supelco, a part of Sigma-Aldrich Corporation.

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### Bibliographie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Rygol B, Kyrcc-Krzemien S, Pajak J et al. Tryptase- and chymase-positive mast cells as a possible prognostic factor in patients with Hodgkin's lymphoma. POLSKIE ARCHIWUM MEDYCYN WEWNETRZNEJ (Polish Archive of Domestic Medicine) 2007; 117:1-2.
5. Anaf V, Chapron C, El Nakadi I et al. Pain, mast cells, and nerves in peritoneal, ovarian, and deep infiltrating endometriosis. Fertility and Sterility. 2006; 86(5):1336-1343.

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### Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Rygol B, Kyrcc-Krzemien S, Pajak J et al. Tryptase- and chymase-positive mast cells as a possible prognostic factor in patients with Hodgkin's lymphoma. POLSKIE ARCHIWUM MEDYCYNY WEWNETRZNEJ (Polish Archive of Domestic Medicine) 2007; 117:1-2.
5. Anaf V, Chapron C, El Nakadi I et al. Pain, mast cells, and nerves in peritoneal, ovarian, and deep infiltrating endometriosis. Fertility and Sterility. 2006; 86(5):1336-1343.

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## Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Rygol B, Kyrz-Krzemien S, Pajak J et al. Tryptase- and chymase-positive mast cells as a possible prognostic factor in patients with Hodgkin's lymphoma. POLSKIE ARCHIWUM MEDYCZYNY WEWNETRZNEJ (Polish Archive of Domestic Medicine) 2007; 117:1–2.
5. Anaf V, Chapron C, El Nakadi I et al. Pain, mast cells, and nerves in peritoneal, ovarian, and deep infiltrating endometriosis. Fertility and Sterility. 2006; 86(5):1336–1343.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Rygol B, Kyrz-Krzemien S, Pajak J et al. Tryptase- and chymase-positive mast cells as a possible prognostic factor in patients with Hodgkin's lymphoma. POLSKIE ARCHIWUM MEDYCYN Y WEWNETRZNEJ (Polish Archive of Domestic Medicine) 2007; 117:1–2.
5. Anaf V, Chapron C, El Nakadi I et al. Pain, mast cells, and nerves in peritoneal, ovarian, and deep infiltrating endometriosis. Fertility and Sterility. 2006; 86(5):1336–1343.

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### Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Rygol B, Kyrz-Krzemien S, Pajak J et al. Tryptase- and chymase-positive mast cells as a possible prognostic factor in patients with Hodgkin's lymphoma. POLSKIE ARCHIWUM MEDYCYN WEWNETRZNEJ (Polish Archive of Domestic Medicine) 2007; 117:1-2.
5. Anaf V, Chapron C, El Nakadi I et al. Pain, mast cells, and nerves in peritoneal, ovarian, and deep infiltrating endometriosis. Fertility and Sterility. 2006; 86(5):1336-1343.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Rygol B, Kyrz-Krzemien S, Pajak J et al. Tryptase- and chymase-positive mast cells as a possible prognostic factor in patients with Hodgkin's lymphoma. POLSKIE ARCHIWUM MEDYCYNY WEWNETRZNEJ (Polish Archive of Domestic Medicine) 2007; 117:1-2.
5. Anaf V, Chapron C, El Nakadi I et al. Pain, mast cells, and nerves in peritoneal, ovarian, and deep infiltrating endometriosis. Fertility and Sterility. 2006; 86(5):1336-1343.

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ProClin™ 950 er et varemærke tilhørende Supelco, en del af Sigma-Aldrich Corporation.