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Catalog No: PA0258

Intended Use

This reagent is for in vitro diagnostic use.

Muscle Specific Actin (HHF35) monoclonal antibody is intended to be used for the qualitative identification by light microscopy of human muscle specific alpha- and gamma-actin isomers in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using BOND Reagents" in your BOND user documentation). Muscle Specific Actin (HHF35) primary antibody is a ready to use product that has been specifically optimized for use with Bond Polymer Refine Detection. The demonstration of human muscle specific alpha- and gamma-actin isomers is achieved by first, allowing the binding of Muscle Specific Actin (HHF35) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system), reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

Reagents Provided

Muscle Specific Actin (HHF35) is a mouse anti-human monoclonal antibody produced as a tissue culture supernatant, and supplied in Tris buffered saline with carrier protein, containing 0.35% ProClin™ 950 as a preservative.

Total volume = 7 mL.

Clone

HHF35.

Immunogen

Sodium dodecylsulphate extracted protein fraction of human myocardium.

Specificity

Human muscle specific alpha- and gamma-actin isomers. Reactive with alpha-actin from skeletal, cardiac and smooth muscle sources. Does not react with non-muscle actin, beta or non-smooth muscle gamma-actin isomers.

Subclass

IgG1.

Total Protein Concentration

Approx 10 mg/mL.

Antibody Concentration

Greater than or equal to 0.07 mg/L as determined by ELISA.

Dilution and Mixing

Muscle Specific Actin (HHF35) primary antibody is optimally diluted for use on the BOND system (includes Leica BOND-MAX system and Leica BOND-III system). Reconstitution, mixing, dilution or titration of this reagent is not required.

Materials Required But Not Provided

Refer to "Using BOND Reagents" in your BOND user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using the BOND system (includes Leica BOND-MAX system and Leica BOND-III system).

Storage and Stability

Store at 2-8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of Muscle Specific Actin (HHF35) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2-8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user¹.

Precautions

- This product is intended for *in vitro* diagnostic use.
- The concentration of ProClin™ 950 is 0.35%. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' Web site, www.LeicaBiosystems.com.

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions². Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

Instructions for Use

Muscle Specific Actin (HHF35) primary antibody was developed for use on the automated BOND system (includes Leica BOND-MAX system and Leica BOND-III system) in combination with Bond Polymer Refine Detection. The recommended staining protocol for Muscle Specific Actin (HHF35) primary antibody is IHC Protocol F. No Pretreatment is recommended.

Results Expected

Normal Tissues

Clone HHF35 stained skeletal and cardiac muscle as well as smooth muscle around blood vessels and in the stroma of a variety of normal tissues. Staining of myofibroblasts in colon, alveolar macrophages, mesenchymal cells, mesothelial cells in lung and Leydig cells in testis was also seen (n=82).

Tumor Tissues

Clone HHF35 stained 63/73 leiomyosarcomas. Weak positivity was noted in 1/1 mucinous cystadenocarcinoma of the ovary and 1/1 rectal MALT lymphoma. No staining except smooth muscle around blood vessels and in stroma was seen in a variety of additional tumors (n=42).

Muscle Specific Actin (HHF35) is recommended for the detection of muscle specific actin in normal and neoplastic tissues, as an adjunct to conventional histopathology using non-immunologic histochemical stains.

Product Specific Limitations

Muscle Specific Actin (HHF35) has been optimized at Leica Biosystems for use with Bond Polymer Refine Detection and BOND ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

Further Information

Further information on immunostaining with BOND reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using BOND Reagents" in your BOND user documentation.

Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Andino L, Cagle PT, Murer B, et al. Pleuropulmonary desmoid tumours: immunohistochemical comparison with solitary fibrous tumours and assessment of β -catenin and cyclin D1 expression. Archives of Pathology and Laboratory Medicine. 2006; 130(10):1503-1509.

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Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Andino L, Cagle PT, Murer B, et al. Pleuropulmonary desmoid tumours: immunohistochemical comparison with solitary fibrous tumors and assessment of β -catenin and cyclin D1 expression. Archives of Pathology and Laboratory Medicine. 2006; 130(10):1503-1509.

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Leica Biosystems Newcastle Ltd