# Bond™ Ready-to-Use Primary Antibody CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4)

**Catalog No: PA0424**

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## **Instructions for Use**

Please read before using this product.

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## **Check the integrity of the packaging before use.**

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Kontroller, at pakken er ubeskadiget før brug.

# Bond™ Ready-To-Use Primary Antibody CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4)

## Catalog No: PA0424

### Intended Use

*This reagent is for in vitro diagnostic use.*

CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4) monoclonal antibody is intended to be used for the qualitative identification by light microscopy of Sialyl Lewis<sup>a</sup> in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using the automated Bond™ system.

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

### Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using Bond Reagents" in your Bond user documentation). CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4) primary antibody is a ready to use product that has been specifically optimized for use with Bond Polymer Refine Detection. The demonstration of Sialyl Lewis<sup>a</sup> is achieved by first, allowing the binding of CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with the automated Bond system, reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

### Reagents Provided

CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4) is a mouse anti-human monoclonal antibody produced as a purified tissue culture supernatant, and supplied in Tris buffered saline with carrier protein, containing 0.35% ProClin™ 950 as a preservative.

Total volume = 7 mL.

### Clone

C241:5:1:4.

### Immunogen

Human colorectal adenocarcinoma cell line COLO 205.

### Specificity

Sialyl Lewis<sup>a</sup>-containing glycolipids. The epitope recognised is designated CA19-9. There is minimal crossreactivity (<1%) with Sialyl Lacto-N-tetraosylceramide. Does not crossreact with Lewis<sup>s</sup>, Lewis<sup>b</sup> or structurally related substances.

### Subclass

IgG1.

### Total Protein Concentration

Approx 10 mg/mL.

### Antibody Concentration

Greater than or equal to 0.35 mg/L as determined by ELISA.

### Dilution and Mixing

CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4) primary antibody is optimally diluted for use on the Bond system. Reconstitution, mixing, dilution or titration of this reagent is not required.

### Materials Required But Not Provided

Refer to "Using Bond Reagents" in your Bond user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using the Bond system.

### Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of CA19-9 (Sialyl Lewis<sup>a</sup>) (C241:5:1:4) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user<sup>1</sup>.

### Precautions

- This product is intended for in vitro diagnostic use.
- The concentration of ProClin™ 950 is 0.35%. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' Web site, [www.LeicaBiosystems.com](http://www.LeicaBiosystems.com).
- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions<sup>2</sup>. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.

- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

## Instructions for Use

CA19-9 (Sialyl Lewis<sup>x</sup>) (C241:5:1:4) primary antibody was developed for use on the automated Bond system in combination with Bond Polymer Refine Detection. The recommended staining protocol for CA19-9 (Sialyl Lewis<sup>x</sup>) (C241:5:1:4) primary antibody is IHC Protocol F. Heat induced epitope retrieval is recommended using Bond Epitope Retrieval Solution 1 for 20 minutes.

## Results Expected

### Normal Tissues

Clone C241:5:1:4 stained ductal epithelium of the breast, pancreas, salivary gland and bile ducts. Staining was also seen in the epithelium of stomach, small intestine, colon, prostate and the squamous epithelium of the esophagus, cervix and tonsil. CA19-9 was also demonstrated in the sebaceous glands of skin, pneumocytes of lung, Hassall's corpuscles and medullary cells of thymus. (Total number of cases stained = 68).

### Tumor Tissues

Clone C241:5:1:4 stained 1/2 colon adenocarcinomas, 1/1 rectum adenocarcinoma, 2/2 stomach adenocarcinomas, 2/2 pancreas adenocarcinomas, 1/2 prostate adenocarcinomas, 2/2 esophagus squamous cell carcinomas, 2/2 larynx squamous cell carcinomas, 1/1 lung non small cell carcinoma, 1/2 uterine invasive squamous cell carcinomas, 1/2 thyroid gland papillary carcinomas, 1/2 thyroid gland follicular carcinomas, 1/2 liver cholangiocarcinomas, 1/1 liver combined hepatocellular cholangiocarcinoma, 1/2 breast infiltrating ductal carcinomas, 1/2 kidney renal cell carcinomas, 1/2 lymph node metastatic carcinomas and 0/23 other tumors. (Total number of cases stained = 52).

CA19-9 (Sialyl Lewis<sup>x</sup>) (C241:5:1:4) is recommended for demonstration of CA19-9 (Sialyl Lewis<sup>x</sup>) in normal and neoplastic tissues.

## Product Specific Limitations

CA19-9 (Sialyl Lewis<sup>x</sup>) (C241:5:1:4) has been optimized at Leica Biosystems for use with Bond Polymer Refine Detection and Bond ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

## Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

## Further Information

Further information on immunostaining with Bond reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using Bond Reagents" in your Bond user documentation.

## Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741–745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369–375.
6. Okubo T and Okabe H. Carbohydrate antigen 19-9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471–476.  
ProClin<sup>®</sup> 950 is a trademark of Supelco, a part of Sigma-Aldrich Corporation.

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## Bibliographie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741–745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369–375.
6. Okubo T and Okabe H. Carbohydrate antigen 19-9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471–476.  
ProClin<sup>®</sup> 950 est une marque commerciale de Supelco, membre du groupe Sigma-Aldrich Corporation.

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### **Bibliografia**

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741-745.
5. Comeselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369-375.
6. Okubo T and Okabe H. Carbohydrate antigen 19-9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471-476.  
ProClin- 950 è un marchio di fabbrica di Supelco, società del gruppo Sigma-Aldrich.

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## Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741–745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369–375.
6. Okubo T and Okabe H. Carbohydrate antigen 19-9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471–476.  
ProCin<sup>®</sup> 950 ist eine Marke von Supelco, einem Unternehmen der Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741–745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369–375.
6. Okubo T and Okabe H. Carbohydrate antigen 19–9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471–476.

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## Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741–745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369–375.
6. Okubo T and Okabe H. Carbohydrate antigen 19–9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471–476.  
ProCin<sup>®</sup> 950 é uma marca registada de Supelco, parte da Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741–745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369–375.
6. Okubo T and Okabe H. Carbohydrate antigen 19-9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471–476.  
ProClin<sup>®</sup> 950 är ett varumärke tillhörande Supelco som ingår i Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741–745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369–375.
6. Okubo T and Okabe H. Carbohydrate antigen 19-9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471–476.  
Το ProCln- 950 είναι εμπορικό σήμα της Supelco, τμήμα της Sigma-Aldrich Corporation.

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## Bibliografi

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
4. Hisa T, Nobukawa B, Suda K, et al. Intraductal carcinoma with complex fusion of tubular glands without macroscopic mucus in main pancreatic duct: dilemma in classification. Pathology International. 2007; 57:741-745.
5. Cameselle-Teijeiro J, Abdulkader I, Soares P, et al. Cystic Tumor of the Atrioventricular Node of the Heart Appears to Be the Heart Equivalent of the Solid Cell Nests (Ultimobranchial Rests) of the Thyroid. American Journal Clinical Pathology 2005;123:369-375.
6. Okubo T and Okabe H. Carbohydrate antigen 19-9 expression by folliculo-tubular structures of normal and neoplastic pituitary. Acta Neuropathologica. 1997; 93:471-476.  
ProClin 950 er et varemærke tilhørende Supelco, en del af Sigma-Aldrich Corporation.

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09 december 2013

Leica Biosystems Newcastle Ltd  
Balliol Business Park West  
Benton Lane  
Newcastle Upon Tyne NE12 8EW  
United Kingdom  
☎ +44 191 215 4242