Bond™ Ready-to-Use Primary Antibody MSH6 (Mismatch Repair Protein) (PU29)

Catalog No: PA0597

Leica Biosystems Newcastle Ltd
Balliol Business Park West
Benton Lane
Newcastle Upon Tyne NE12 8EW
United Kingdom
☎ +44 191 215 4242

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Instructions for Use

Please read before using this product.

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Check the integrity of the packaging before use.

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Kontroller, at pakken er ubeskadiget før brug.

Bond™ Ready-To-Use Primary Antibody MSH6 (Mismatch Repair Protein) (PU29)

Catalog No: PA0597

Intended Use

This reagent is for in vitro diagnostic use.

MSH6 (Mismatch Repair Protein) (PU29) monoclonal antibody is intended to be used for the qualitative identification by light microscopy of human mismatch repair protein (MSH6) in formalin-fixed, paraffin-embedded tissue by immunohistochemical staining using an automated Bond™ system.

The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using Bond Reagents" in your Bond user documentation). MSH6 (Mismatch Repair Protein) (PU29) primary antibody is a ready to use product that has been specifically optimized for use with Bond Polymer Refine Detection. The demonstration of human mismatch repair protein (MSH6) is achieved by first, allowing the binding of MSH6 (Mismatch Repair Protein) (PU29) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with an automated Bond system, reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

Reagents Provided

MSH6 (Mismatch Repair Protein) (PU29) is a mouse anti-human monoclonal antibody produced as a tissue culture supernatant, supplied in Tris buffered saline with animal serum and carrier protein, containing 0.35% ProClin™ 950 as a preservative.

Total volume = 7 mL.

Clone

PU29.

Immunogen

Prokaryotic recombinant protein corresponding to a 359 amino acid portion of the human MSH6 molecule.

Specificity

Human mismatch repair protein (MSH6).

Subclass

IgG1.

Total Protein Concentration

Approx 10 mg/mL.

Antibody Concentration

Greater than or equal to 10.67 mg/L as determined by ELISA.

Dilution and Mixing

MSH6 (Mismatch Repair Protein) (PU29) primary antibody is optimally diluted for use on a Bond system. Reconstitution, mixing, dilution or titration of this reagent is not required.

Materials Required But Not Provided

Refer to "Using Bond Reagents" in your Bond user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using a Bond system.

Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of MSH6 (Mismatch Repair Protein) (PU29) are: turbidity of the solution, odor development, and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user¹.

Precautions

- This product is intended for *in vitro* diagnostic use.
- The concentration of ProClin™ 950 is 0.35%. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' web site, www.LeicaBiosystems.com.

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions². Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Retrieval, incubation times or temperatures other than those specified may give erroneous results. Any such change must be validated by the user.

Instructions for Use

MSH6 (Mismatch Repair Protein) (PU29) primary antibody was developed for use on an automated Bond system in combination with Bond Polymer Refine Detection. The recommended staining protocol for MSH6 (Mismatch Repair Protein) (PU29) primary antibody is IHC Protocol F. Heat induced epitope retrieval is recommended using Bond Epitope Retrieval Solution 1 for 40 minutes.

Results Expected

Normal Tissues

Clone PU29 detected the MSH6 protein expressed in epithelial cell nuclei of lung, skin, cervix, breast, thyroid, salivary gland, and throughout the GI tract. Lymphocytes and macrophages were also stained in a variety of tissue types. Other positive staining included bone marrow blast cells, GI tract, breast and cervical stroma, Leydig cells and spermatogonia of testes. (Total number of cases stained = 31).

Tumor Tissues

Clone PU29 detected the MSH6 protein in 54/58 colon tumors. Clone PU29 also detected the MSH6 protein in lymphomas (9/11), sarcomas (7/7), ovarian tumors (7/7), lung tumors (6/7), breast tumors (4/5), stomach tumors (4/4), endometrial tumors (4/4), neuroendocrine tumors (2/4), BPH/prostate tumors (3/3), melanomas (3/3), adrenal tumors (3/3), germ cell tumors (3/3), bladder tumors (3/3), squamous cell carcinomas (2/3), liver tumors (1/3), pancreatic tumors (2/2), thyroid tumors (2/2), a small bowel tumor (1/1), a skin tumor (1/1), a thymus tumor (1/1), a neural tumor (0/1) and kidney tumors (0/4). (Total number of cases stained = 140).

MSH6 (Mismatch Repair Protein) (PU29) is recommended for use in detecting MSH6 protein expression in normal and neoplastic tissues, particularly to aid in the identification of the loss of MSH6 expression in hereditary non-polyposis colon cancer and sporadic cancers.

Product Specific Limitations

MSH6 (Mismatch Repair Protein) (PU29) has been optimized at Leica Biosystems for use with Bond Polymer Refine Detection and Bond ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

Further Information

Further information on immunostaining with Bond reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using Bond Reagents" in your Bond user documentation.

Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.

Leica Biosystems and logo design are trademarks of Leica Biosystems A.G and are used under licence. © 2007 Leica Biosystems Newcastle Ltd. All rights reserved.

ProClin™ 950 is a trademark of Supelco, a part of Sigma-Aldrich Corporation.

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Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
 2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
 3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
- Leica Biosystems und das Logo sind Marken der Leica Biosystems AG und werden unter Lizenz benutzt. © 2007 Leica Biosystems Newcastle Ltd. Alle Rechte vorbehalten.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
 2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
 3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
- Leica Biosystems och dess logotypdesign är varumärken tillhörande Leica Biosystems A.G och används under licens. © 2007 Leica Biosystems Newcastle Ltd. Alla rättigheter reserverade.

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Bibliografi

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. Leica Biosystems og logodesign er varemærker tilhørende Leica Biosystems A.G og anvendes under licens. © 2007 Leica Biosystems Newcastle Ltd. Alle rettigheder forbeholdes. ProClin™ 950 er et varemærke tilhørende Supelco, en del af Sigma-Aldrich Corporation.

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16. September 2009

Leica Biosystems Newcastle Ltd
Balliol Business Park West
Benton Lane
Newcastle Upon Tyne NE12 8EW
United Kingdom
☎ +44 191 215 4242