Bond™ Ready-to-Use Negative Control Negative (Rabbit)

Catalog No: PA0777

Leica Biosystems Newcastle Ltd
Balliol Business Park West
Benton Lane
Newcastle Upon Tyne NE12 8EW
United Kingdom
J +44 191 215 4242

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Instructions for Use

Please read before using this product.

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Check the integrity of the packaging before use.

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Catalog No: PA0777

Intended Use

This reagent is for in vitro diagnostic use.

Negative (Rabbit) is intended to be used as a rabbit negative control for qualitative immunohistochemical staining using an automated Bond™ system. Negative (Rabbit) is not directed against a known antigen in human tissue. The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Summary and Explanation

Immunohistochemical techniques can be used to demonstrate the presence of antigens in tissue and cells (see "Using Bond Reagents" in your Bond user documentation). Negative (Rabbit) is a ready to use product that has been specifically optimized for use with Bond Polymer Refine Detection.

In some tissues, non-specific binding may occur, especially in neoplastic or necrotic tissue. The use of Negative (Rabbit) is recommended to aid in the identification of cells, tissues or tissue components, which may non-specifically bind rabbit antibodies and will allow better interpretation of specific staining at the antigenic site.

The demonstration of non-specific staining is achieved by first, allowing the binding of Negative (Rabbit) to the section, and then visualizing this binding using the reagents provided in the detection system. The use of these products, in combination with an automated Bond system, reduces the possibility of human error and inherent variability resulting from individual reagent dilution, manual pipetting and reagent application.

Reagents Provided

Negative (Rabbit) is purified rabbit immunoglobulin supplied in Tris buffered saline with carrier protein, containing 0.35% ProClin™ 950 as a preservative.

Total volume = 7 mL.

Clone

Not applicable.

Immunogen

Not applicable.

Specificity

Not applicable.

Subclass

Not applicable.

Total Protein Concentration

Approx 10 mg/mL.

Antibody Concentration

Greater than or equal to 1 mg/L.

Dilution and Mixing

Negative (Rabbit) is optimally diluted for use on a Bond system. Reconstitution, mixing, dilution or titration of this reagent is not required.

Materials Required But Not Provided

Refer to "Using Bond Reagents" in your Bond user documentation for a complete list of materials required for specimen treatment and immunohistochemical staining using a Bond system.

Storage and Stability

Store at 2–8 °C. Do not use after the expiration date indicated on the container label.

The signs indicating contamination and/or instability of Negative (Rabbit) are: turbidity of the solution, odor development and presence of precipitate.

Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user¹.

Precautions

- This product is intended for in vitro diagnostic use.
- The concentration of ProClin™ 950 is 0.35%. It contains the active ingredient 2-methyl-4-isothiazolin-3-one, and may cause irritation to the skin, eyes, mucous membranes and upper respiratory tract. Wear disposable gloves when handling reagents.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or regional office of Leica Biosystems, or alternatively, visit the Leica Biosystems' Web site, www.LeicaBiosystems.com.

- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions². Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, State or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.

Instructions for Use

Negative (Rabbit) was developed for use on an automated Bond system in combination with Bond Polymer Refine Detection. The recommended staining protocol for Negative (Rabbit) is IHC Protocol F. Negative (Rabbit) should be used in place of the rabbit primary antibody with each patient specimen, a separate section should be stained for each retrieval method employed (including no retrieval).

Results Expected

Normal Tissues

Negative (Rabbit) gave no staining in a range of normal tissues. (Number of cases stained = 55).

Negative (Rabbit) is recommended for use as a negative control for Bond Ready-to-Use rabbit primary antibodies.

Product Specific Limitations

Negative (Rabbit) has been optimized at Leica Biosystems for use with Bond Polymer Refine Detection and Bond ancillary reagents. Users who deviate from recommended test procedures must accept responsibility for interpretation of patient results under these circumstances. The protocol times may vary, due to variation in tissue fixation and the effectiveness of antigen enhancement, and must be determined empirically. Negative reagent controls should be used when optimizing retrieval conditions and protocol times.

Troubleshooting

Refer to reference 3 for remedial action.

Contact your local distributor or the regional office of Leica Biosystems to report unusual staining.

Further Information

Further information on immunostaining with Bond reagents, under the headings Principle of the Procedure, Materials Required, Specimen Preparation, Quality Control, Assay Verification, Interpretation of Staining, Key to Symbols on Labels, and General Limitations can be found in "Using Bond Reagents" in your Bond user documentation.

Bibliography

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin[®] 950 is a trademark of Supelco, a part of Sigma-Aldrich Corporation.

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Bibliographie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin[®] 950 est une marque commerciale de Supelco, membre du groupe Sigma-Aldrich Corporation.

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Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin™ 950 è un marchio di fabbrica di Supelco, società del gruppo Sigma-Aldrich.

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Bibliografie

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
 2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
 3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996.
- ProClin™ 950 ist eine Marke von Supelco, einem Unternehmen der Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin[®] 950 es una marca comercial de Supelco, parte de Sigma-Aldrich Corporation.

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Bibliografia

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin[®] 950 é uma marca comercial de Supelco, parte da Sigma-Aldrich Corporation.

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1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. Το ProClim™ 950 είναι εμπορικό σήμα της Supelco, τμήμα της Sigma-Aldrich Corporation.

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Bibliografi

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.
3. Bancroft JD and Stevens A. Theory and Practice of Histological Techniques. 4th Edition. Churchill Livingstone, New York. 1996. ProClin[®] 950 er et varemærke tilhørende Supelco, en del af Sigma-Aldrich Corporation.

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13. februar 2009

Leica Biosystems Newcastle Ltd
Balliol Business Park West
Benton Lane
Newcastle Upon Tyne NE12 8EW
United Kingdom
☎ +44 191 215 4242